

REMEDI REMEX-T100

QUICK USER GUIDE



Turn the unit ON/OFF

Hold down 3 seconds to turn ON.
The unit will turn OFF automatically
after 2 minutes of no use.

Adjust exposure time

Use UP/DOWN-keys to scroll thru the preset settings.
Hold down both UP/DOWN-keys to change from adult to child settings.
Hold down UP/DOWN-keys again to change to manual settings.

Exposure

Hit and hold down to make an exposure. Notice that unit makes alert beeps before the actual exposure happens. Radiation is ON when the cone light illuminates.



NOTE!

**PLEASE DO NOT LEAVE THE X-RAY TO A CHARGER OVERNIGHT.
CHARGE THE UNIT MAX 3 HOURS AT THE TIME**

Familiarize the usage of handheld x-ray before using with patients
The Operation Manual is in the USB stick in the envelope

any questions, please call
(800) 484-2295



PHOENIX IMAGING

freeing you from the wall